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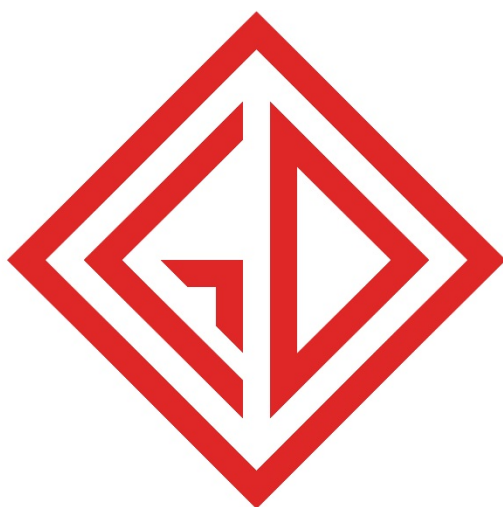
IMPORTANT SAFEGUARDS

The  symbol on the product or on its packaging indicates that this product may not be treated as household waste. Instead it shall be handed over to the applicable collection point for the recycling of electrical and electronic equipment. By ensuring this product is disposed of correctly, you will help prevent potential negative consequences for the environment and human health, which could otherwise be caused by inappropriate waste handling of this product.

For more detailed information about recycling of this product, please contact your local city office, your household disposal service or the shop where you purchased the product.

Read the instruction booklet carefully.

Before using the appliance, check that all the filters are in place and that the power supply is the same indicated on rating label.

Check the filters regularly and, if necessary, brush them or replace them.

If necessary, wash the filters. Allow the filters to dry completely before replacing them.

Never position the vacuum nozzle near the eyes, ears or nose when the appliance is in operating.

Do not use the appliance to vacuum burning objects and water.

Always remove the plug from the power socket when the appliance is not in use and before carrying out any maintenance or cleaning.

Do not switch on the appliance if it appears to malfunction in any way

Do not run the vacuum cleaner over the power cable, as this might damage the insulating sheath.

Never pull the power cable or the appliance itself to remove the plug from the socket.

Do not allow children or the infirm to use the appliance

This appliance is designed exclusive for

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domestic use as describen in the instruction booklet and not use the appliance outside or on wet surface.

Don't wet the appliance and the accessories for cleaning , neither partially.

Please verify regularly the flexible and not use if damaged.

Don not use the appliance if the dust collector is blocked, this may cause a possible motor damage.

For all repairs contact your local dealer or an authorized Service Center.

The manufacture reserves the right to modify the appliance and the relative accessories without prior notice. In order to maintain the efficiency of the appliance and to comply with the requirements of the warranty valid for 24 month from the buy-date, always insist on original spare parts and accessories.

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PRODOTTO: ASPIRAPOLVERE MODELLO- TIPO: LABO I – II -III
PRODUCT: CLEANER MODEL- TYPE: LABO I-II-III
PRODUIT: ASPIRATEUR MODELE-TYPE: LABO I-II-III

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Declares under its responsibility that the machine is complies with directives EC, and subsequent modifications, and the standards EN:

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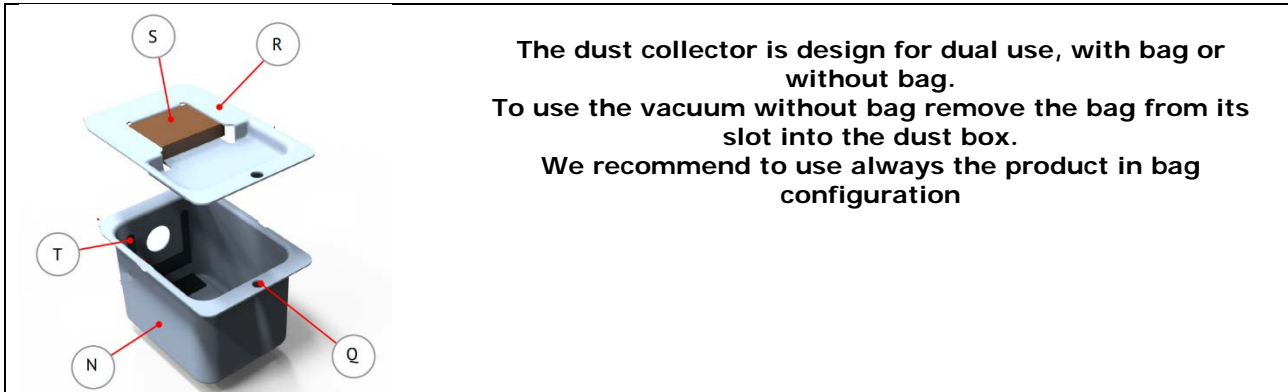
ASSEMBLY

Before using the appliance make sure that all the filters are in the correct position.

Make sure that the micro por bag is properly fixed into the dust box and all filter is properly fixed.

Fit the hose into the hose inlet and rotate clockwise to lock

Connect the power cord to the power socket 6.

MAINTENANCE

Cleaning and maintenance of the filters.

Do any maintenance operation always with the product switched off and the power cord disconnected from power socket.

The dust bag must be regularly replaced according to the frequency of use and type of dust .

The cartridge filter is recommended to be replace every 4/6 month, also in relation to the frequency of use and type of dust.

clean under running water or brush the filter cartridge periodically with a soft brush to avoid damaging the filtering surface. It is advisable to replace the filter at least once every 4/6 months and in any case according to the frequency of use and the type of dirt sucked.

Check the filling status of the bag replacing it when necessary.

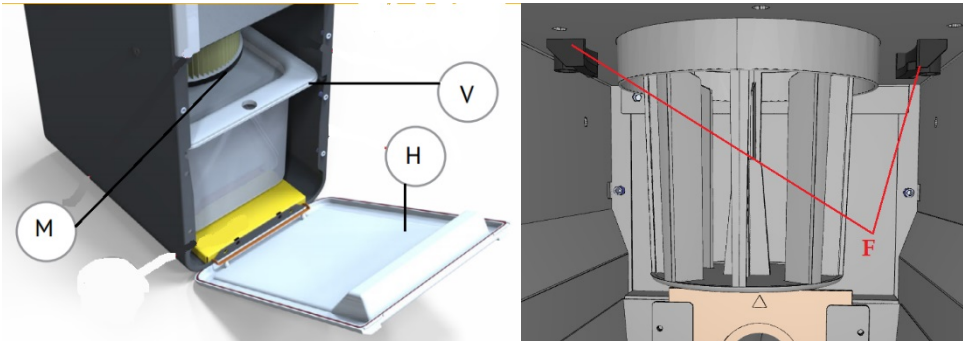
The use of original filter bags is recommended.

Cartridge filter: open the front H panel pulling the handle and remove the dust bag box V .

Turning light the filter M and remove it. Insert the new cartridge filter and turn until it's complete fixation on supports F.

The filter is a washable type. Periodically (about every 1 or months) remove the filter and clean it under running water. Once dry, reposition the filter.


A complete replacement is recommended every 4-6 months.


Dust bag : according with the use and in any case when the full bag indicator is red change the dust bag.

Open the front panel H pulling the handle. Remove the dust bag box V and remove the bag.

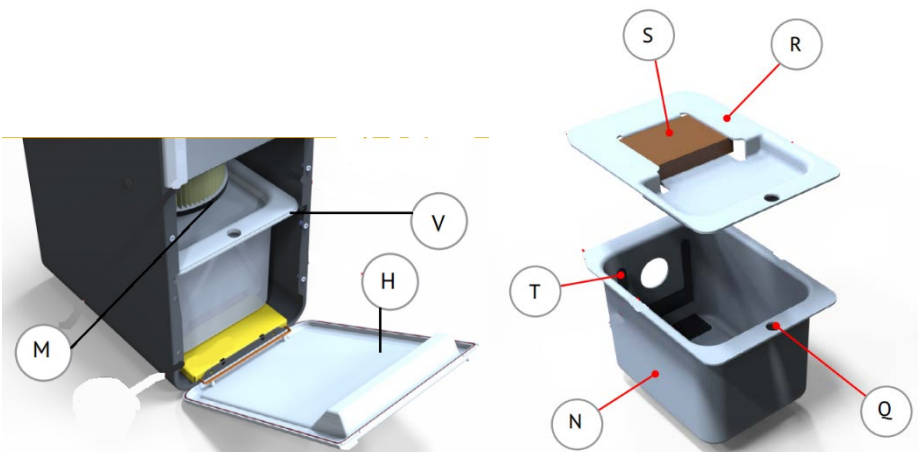
Insert the bag's flange into the support T and restore the box V in it's position into the dust room.

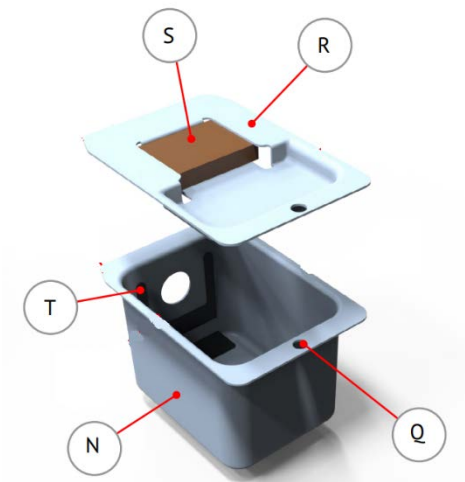Every 4/6 month and in any case in consequence of the use replace the filter S

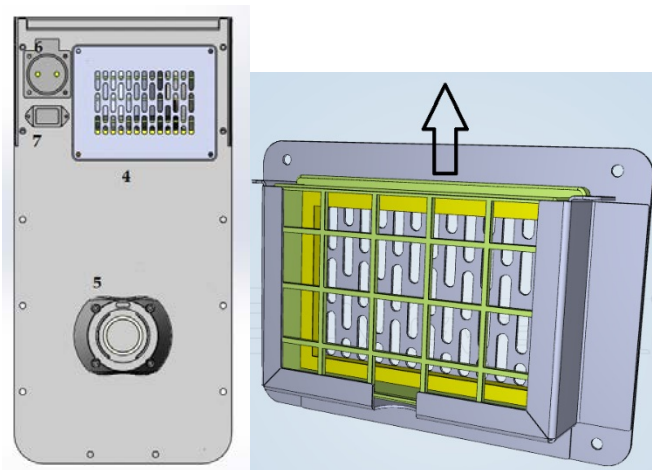
Exhaust heap filter. Unscrew the 4 screws of the grid / support (4) Air positioned on the back of the product and remove the filter frame . Proceed with changing the filter as show in the image and restore the grill in place. Fit the grid with the 4 screws. This filter IS NOT washable one.

USE

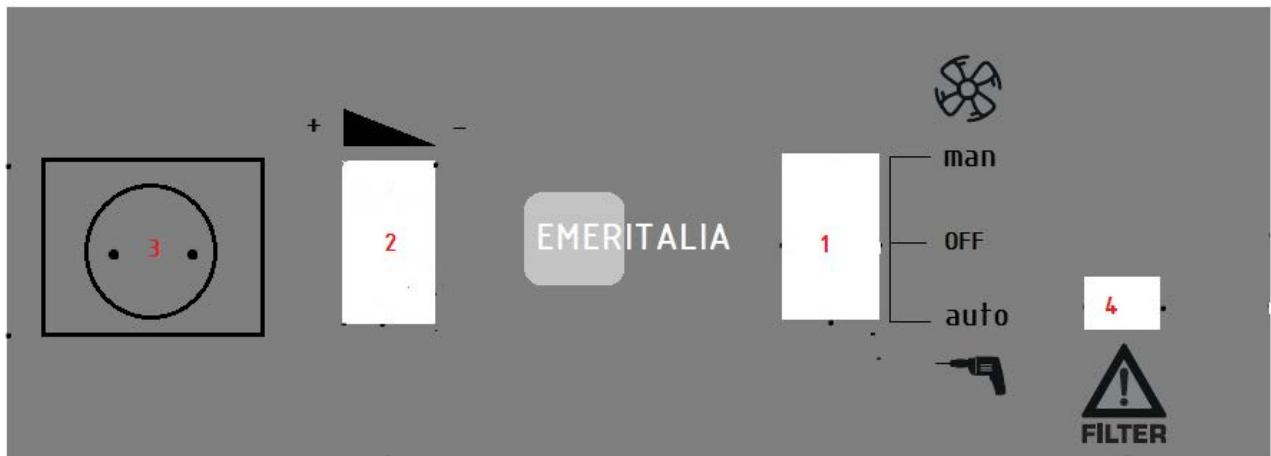Modell LABO EC -ECMM

1. MAIN SWITHC
2. POWER REGULATION
3. TOOL SOCKET
4. FUUL BAG INDICATOR

0 (0) OFF, SWITCH OFF

 (I) ON, MANUALLY OPERATED: VACUUMING

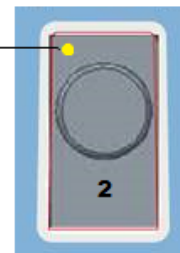
 AUTOMATICALLY OPERATED WITH CONNECTING ELECTRIC POWER TOOL TO SOCKET - SOCKET FOR TOOLS

FOR MODEL EC MAX POWER
INPUT FOR TOOL SOCKET IS
2000W


FOR MODEL EC.MM MAX POWER
INPUT FOR TOOLS SOCKET IS
250W

In the EC.MM model the regulation screw permit the calibration with the micromotor.
Turn right or left the screw for set the input start.

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MODELLO LABO DS- DSMM

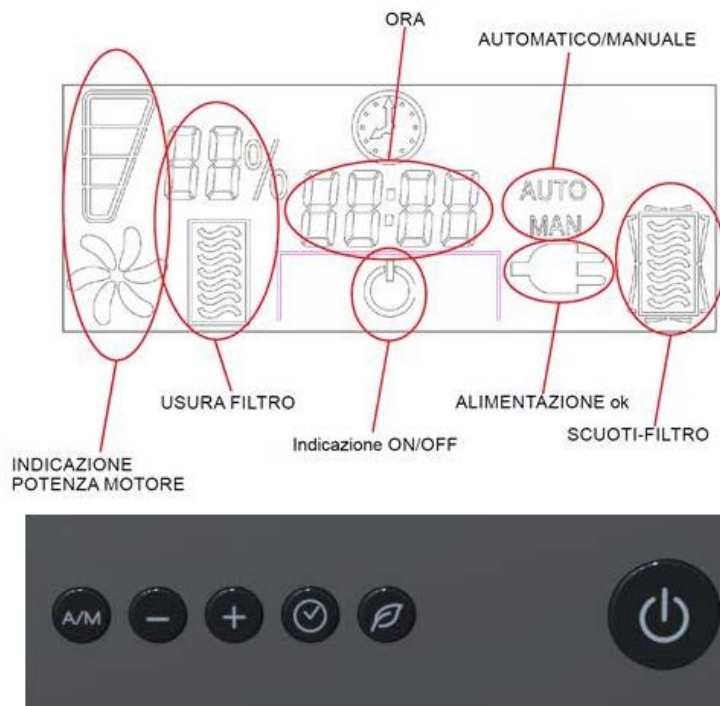ON: main switch

A/M: press to set to use mode – auto or manual

IN AUTOMATIC MODE WHEN THE TOOLS WILL BE SWITCH ON AT THIS SAME TIME ALSO VACUUM WILL RUN, SWITCHING OFF THE TOOLS ALSO THE VACUUM SHUTTING OFF

FOR MODEL EC MAX POWER INPUT FOR TOOL SOCKET IS 2000W

FOR MODEL EC.MM MAX POWER INPUT FOR TOOLS SOCKET IS 250W

+/- : press to set the power motor 25-50-75-100%

CLOCK : CLOCK SET >keep press the button till the digit are lamping > to set the hour digit use + or -> press again the button to set the minute digit> press the button to confirm

LEAF: to clean the filter press the relative button. Anyway the system shake automatically the filter every 4 hours.

MICROMOTOR CALIBRATION

- > KEEP PRESS THE A/M AND LEAF BUTTON AND AT THE SAME TIME INSERT THE POWER CORD
- > WAIT TILL ON DISPLAY SHOW THE WRITTEN "ST 000"
- > WITH + OR – SET THE VALUE FROM 0 TO 250
- > PRESS CLOCK TO CONFIRM

HOUR:

- KEEP PRESS + AND – AND AT THE SAME TIME INSERT THE PLUG
- WAIT TILL ON DISPLAY SHOW "TO / ON 00:00 " .
- TO= TATAL TIME / ON= TIME MOTOR (NB: 1 IS = 10 HOUR)

AIR FLOW PERFORMANCE - $Q = 0.0019 * D^2 * H_{Corr}^{0.5}$

ORIFICE	A	Win	Win Corr.	HA mmH2O	H CORR. mmH2O	Q l/sec.	Wout	EFF. %
48.0	3.95	898	892.1	64	63	34.8	21.6	2.4
40.0	3.95	899	893.1	126	125	33.9	41.5	4.6
30.0	3.91	891	885.2	332	329	31.0	99.9	11.3
23.0	3.80	865	859.4	665	658	25.8	166.4	19.4
19.0	3.70	843	837.5	962	952	21.2	197.6	23.6
16.0	3.57	817	811.7	1236	1223	17.0	204.1	25.1
13.0	3.41	780	774.9	1501	1485	12.4	180.3	23.3
10.0	3.22	736	731.2	1754	1736	7.9	134.8	18.4
6.5	3.01	688	683.5	1969	1948	3.5	67.7	9.9
0.0	2.78	636	631.9	2145	2122	0.0	0.0	0.0

REMARKS : MEASUREMENTS CARRIED OUT AFTER 1', WITH SEALED ORIFICE AFTER 20''

VOLTMETER:NORMA D5135 AMPEROMETER:NORMA D5135 WATTMETER:NORMA D5135

AMBIENT °C : 20 ATM.PRESS. mmHg : 770 RH % : 33

AIR DENSITY RATIO : 1.013

CORRECTION FACTORS IN TERMS OF AIR DENSITY (20°C - RH50% - 760mmHg)

WIN CORRECTION FACTOR : 0.993 DEPRESSURE CORR.FACTOR H : 0.989